



STATE OF OKLAHOMA
BOARD OF TESTS FOR ALCOHOL AND DRUG INFLUENCE

Dr. Kenneth E. Blick, Ph.D.
Chairman

Administrative Office at:
Board of Tests for Alcohol and Drug Influence
Post Office Box 36307
Oklahoma City, Oklahoma 73136-2307

J. Robert Blakeburn
State Director of Tests
for Alcohol and Drug Influence

August 26, 2009

Emmett E. Goodson
Administrative Assistant

**ACTION NO. 09-02 BY THE STATE DIRECTOR OF TESTS FOR ALCOHOL AND DRUG
INFLUENCE**

Under authority of 47 O.S. § 751-761 and OAC 40:1-1-3. and 40:30-1-3. Breath-alcohol analysis with the Intoxilyzer Model 8000; promulgated by the Board of Tests for Alcohol and Drug Influence, the following official ACTION is hereby taken by the State Director of Tests for Alcohol and Drug Influence.

**OPERATING PROCEDURE FOR BREATH-ALCOHOL ANALYSIS
WITH THE MODEL 8000 INTOXILYZER AND EXTERNAL PRINTER**

- (1) The State Director of Tests for Alcohol and Drug Influence hereby prescribes and approves the attached "***OPERATING PROCEDURE FOR BREATH-ALCOHOL ANALYSIS WITH THE MODEL 8000 INTOXILYZER WITH EXTERNAL PRINTER***" for the analysis of alcohol in specimens of breath by means of the Model 8000 Intoxilyzer with external printer, in keeping with provisions of OAC 40:30-1-3 promulgated by the Board of Tests for Alcohol and Drug Influence. The ***OPERATING PROCEDURE*** is attached hereto and is hereby incorporated in this record of official ACTION.
- (2) All testing completed on or after April 16, 2008, that has followed the procedures required by Action 08-04 OR Action 09-02, shall be deemed to have been performed in accordance with the Board of Test rules.
- (3) This ACTION by the State Director of Tests for Alcohol and Drug Influence shall take effect on and after August 27, 2009.

J. Robert Blakeburn
State Director of Tests
for Alcohol and Drug Influence



State of Oklahoma
Board of Tests for Alcohol and Drug Influence
Office of the State Director of Tests for Alcohol and Drug Influence
Post Office Box 36307
Oklahoma City, Oklahoma 73136-0415

Breath-Alcohol tests administered by means of the Model 8000 Intoxilyzer under the provisions of 47 O.S. § 751-761 and OAC 40:30-1-3., June 12, 2008, by the Board of Tests for Alcohol and Drug Influence, shall be performed in accordance with the following **OPERATING PROCEDURE** established by the State Director of Tests for Alcohol and Drug Influence.

**OPERATING PROCEDURE FOR BREATH-ALCOHOL ANALYSIS
WITH THE MODEL 8000 INTOXILYZER WITH EXTERNAL PRINTER**

A. SUBJECT PREPARATION

1. Determine subject's mouth has no presence of any substantial loose material(s), or foreign substance(s), or any such substance(s). Metal, porcelain or hard plastic items need not be removed.
2. Ascertain that, **at least fifteen (15) minutes prior to collection of the first breath specimen**, the subject has not ingested any substance at all, vomited, eaten, or smoked. Such observation may be carried out by the breath-alcohol analysis Operator or Specialist or by any other qualified person.

B. TESTING SEQUENCE

2. Press the Green Start button on the Intoxilyzer and follow the instructions displayed. Remove a new sealed mouthpiece from its clear bag and insert it into the end of the Breath Tube before the first test is administered.
3. Upon completion of a test, the printer will print the testing data on the Officer's Affidavit.
4. In the event an informational message requires administration of additional testing, keep and distribute **all** printouts from the attempted testing with the affidavit.

C. ROUTING OF DOCUMENTS

1. The printer will print two copies. Remove the Affidavit from the external printer and sign the copy to be forwarded to the Department of Public Safety. Make two copies of the FRONT:
2. Forward the signed printout to the Department.
3. Deliver the other printout to the person tested.
4. Deliver one copy to the prosecutor and retain one for your records.
KEEP YOUR COPY.
5. In the event there are no information messages and the internal printer prints results, follow the following procedure:
 - a. Handwrite the affidavit on a blank form, staple the tape or a copy of it to the handwritten form.
 - b. Contact the Board of Tests to print the Intoxilyzer affidavit. When you receive the affidavit from the Board, sign and mail that form to the Department of Public Safety.

Note: improper routing of documents does not invalidate the test. It may result in no action by the Department of Public Safety or the District Attorney.

D. COMPLETION OF TEST

5. Leave the Intoxilyzer and Printer on and ready for the next test.

E. REPETITION OF BLANK, SUBJECT, AND CONTROL TESTS

6. If any other programed message interrupts the test or the breath-alcohol concentrations of the subject's BREATH TEST 1 and BREATH TEST 2 differ by more than 0.03 g/210 L, repeat the entire procedure in Section B. Attach the program message to the affidavit. **DO NOT THROW ANY PRINTOUT AWAY.**

